

# Codonics® Safe Label System® **Technical Brief**

## Pad and Filter Spare Part

### Overview

This document explains how to properly replace the ink pad and filter for the Codonics Safe Label System® (SLS) Point of Care Station (PCS).

### Tools Needed


- ◆ Gloves
- ◆ T10 screwdriver
- ◆ Needle nose pliers
- ◆ Rags
- ◆ 99% alcohol
- ◆ 99% alcohol pads \*
- ◆ New ink pad \*
- ◆ New ink filter \*

\* Provided with spare part kit

### Process

**Caution:** The ink pad and filter in the SLS will be saturated with ink and messy to handle. Wear gloves during this process.


1. Unload labels and remove them from the unit.
2. Properly power down the unit and unplug the power cord.
3. Remove the rear door and set it aside.
4. Lay the unit on its left side and place a rag on the work surface under the fan cover.

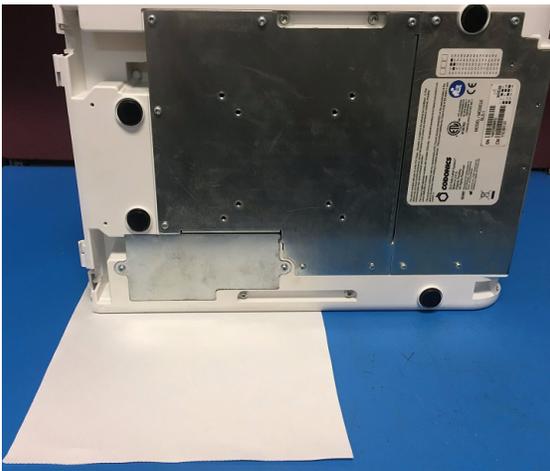


5. Use a T10 screwdriver to remove the two screws securing the fan cover. Save the screws for step 14.




6. Carefully remove the fan cover and lay it on top of the rag.

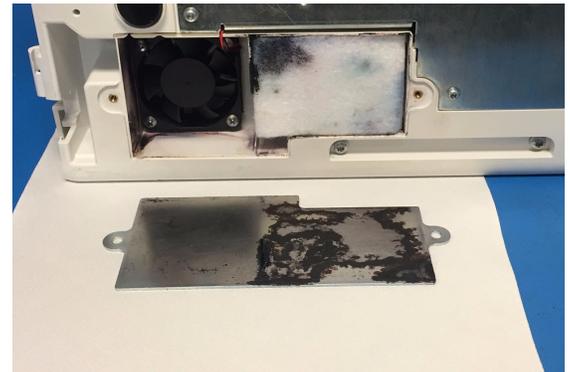


7. Use a rag soaked with 99% alcohol to clean the fan cover.

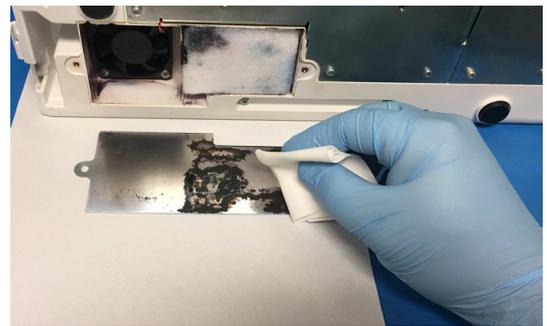


8. The ink pad will be saturated with ink making it messy to handle. Use a pair of needle nose pliers to remove the pad and discard it.

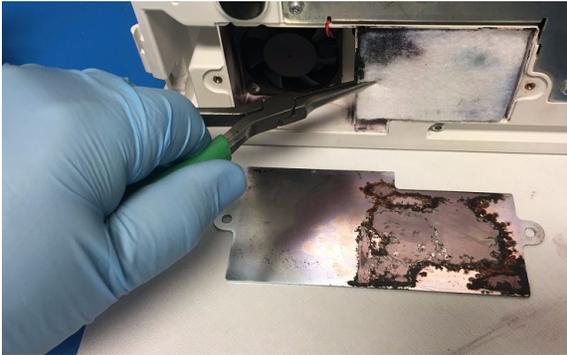



9. The ink filter will be saturated with ink making it messy to handle. Use a pair of needle nose pliers to remove the filter and discard it.

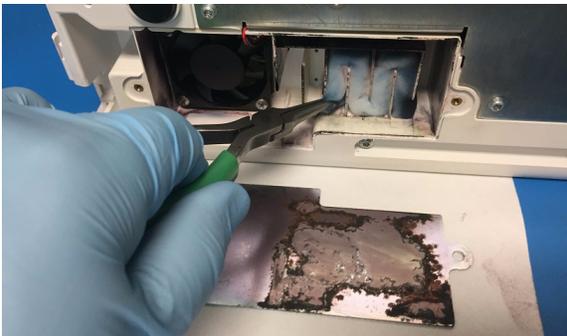


10. Use a rag soaked with 99% alcohol to clean excess ink from the unit chassis.

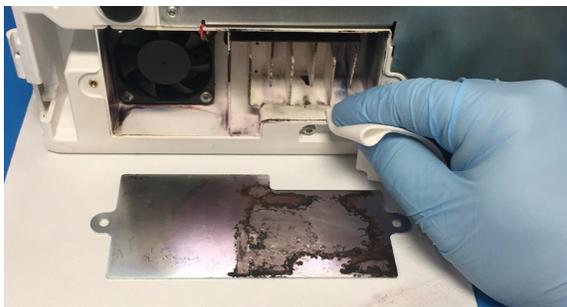


11. Fold the new ink filter in half as shown, and then fold the sides down to form an "M" shape.

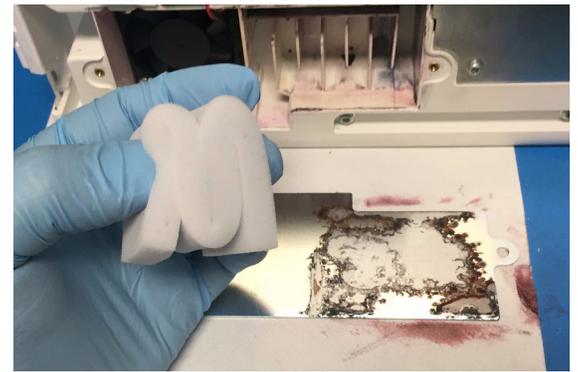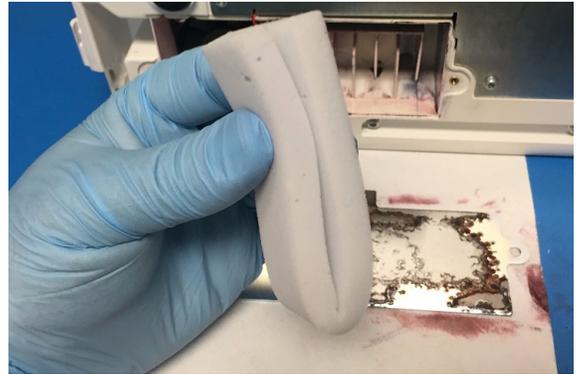

12. Press the new ink filter into the ribs as shown.

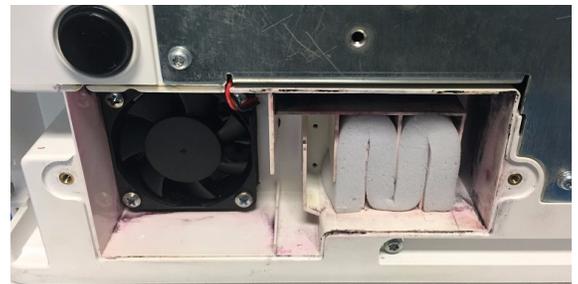

13. Place the new ink pad into place over the filter.

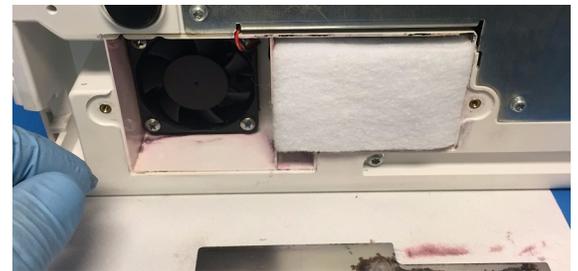

14. Place the fan cover into position and secure with the two screws from step 5 using a T10 screwdriver.



15. The process is complete. Turn the unit upright, replace the rear door, plug it in and reload labels.

### **Technical Support**

If problems occur during replacement, contact Codonics Technical Support at any time.

Phone: +1.440.243.1198

Email: [support@codonics.com](mailto:support@codonics.com)

Website: [www.codonics.com](http://www.codonics.com)

---

*Get it all with just one call*  
**800.444.1198**



All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.  
Patents: [www.codonics.com/ip/patents](http://www.codonics.com/ip/patents).

Copyright © 2020 Codonics, Inc. Printed in the U.S.A. Part No. 901-291-001.03

17991 Englewood Drive  
Cleveland, OH 44130 USA  
+1.440.243.1198  
+1.440.243.1334 Fax  
Email [info@codonics.com](mailto:info@codonics.com)  
[www.codonics.com](http://www.codonics.com)

Codonics Trading Co, Ltd.  
317 Xianxia Rd. Building B  
Unit 1412  
Changning Dist., Shanghai  
P.R. China, 200051  
86-21-62787701  
86-21-62787719 Fax

Codonics Limited KK  
317 Xianxia Rd. Building B  
New Shibaura Bldg. F1  
1-3-11, Shibaura  
Minato-ku, Tokyo, 105-  
0023 JAPAN  
Phone: 81-3-5730-2297  
Fax: 81-3-5730-2295