

Codonics Hand Scanner Holster Installation


Technical Brief

Overview

This document explains how to properly attach the Wired Hand Scanner Holster to the Codonics Safe Label System™ SLS 500i™ shelf with storage compartment.

Location of the Holster

The holster will be located on the outer surface of the 500i shelf opposite the cart as shown.

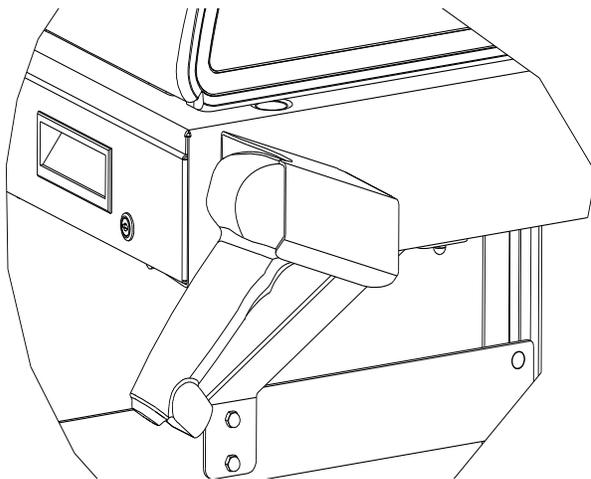


Application of Tape

These steps will show the application of VHB tape to the holster for mounting onto the 500i shelf. The location of the tape depends on which side the 500i shelf is attached to the cart.

For Right Side Installation

Remove the white backing from the VHB tape and apply in the location shown.

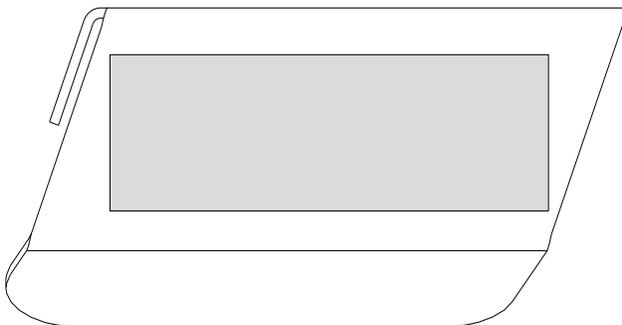


For Left Side Installation

Remove the white backing from the VHB tape and apply the tape in the location shown.



Installation

1. Remove the red and white liner from the VHB tape.
2. Position the holster approximately one inch from the front edge of the 500i shelf. Tilt the holster so that the open end is aligned with the top of the 500i shelf and the closed end is aligned with the bottom of the 500i shelf as shown.

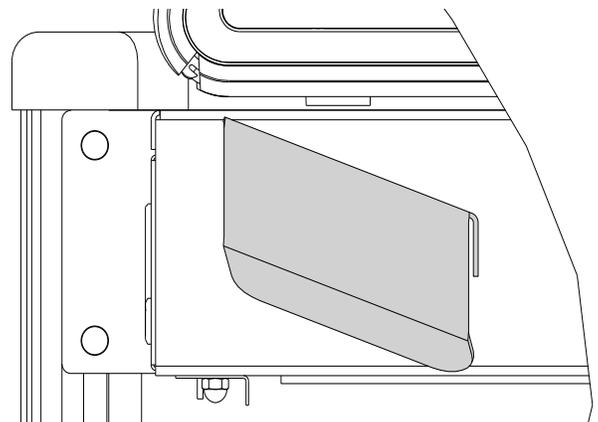

3. Press the holster onto the 500i shelf and apply pressure for 30 seconds to one full minute.

Technical Support

Please contact Codonics Technical Support at any time.

Phone: +1.440.243.1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2013 Codonics, Inc. Printed in the U.S.A. Part No. 901-582-001.01.



17991 Englewood Drive
Middleburg Heights, OH 44130
USA
+1.440.243.1198
+1.440.243.1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023
JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295