

# Batch & Copy Mode

Codonics SLS 550i™ Point of Care Station (PCS) is perfectly suited for pharmacy, OR, ICU and PACU where duplicate labels are often required.

## Batch Mode

In the hospital pharmacy, it is commonplace to prepare medications in batches where multiple copies of the same label are required. Codonics Safe Label System offers Batch Mode, an optional feature that enables up to 999 identical labels to be created from a single scan of a vial or ampoule. Batch Mode labels can accommodate two sets of initials as needed, when one person prepared the label but not the syringe, or when a second check for safety is required. If desired, labels can also include the lot number.

 Phenylephrine  
100 mcg/mL



Batch Mode

### Additional Advantages:

- Audible/visual identification of parenteral vial to prevent drug selection errors
- Fast, easy touch screen entry of the desired number of identical labels to be printed
- Touch screen selection for individually cut or strip-cut labels
- Touch screen display indicates printing status (such as "Printing 10 of 250")
- Optional on-screen entry of lot number
- Configurable per unit



Optional lot number

# Copy Mode


The OR, ICU and PACU often have the need for multiple identical labels for IV applications, where the IV bag, the line, the catheter, and the pump must all be labeled alike. Additionally, multiple copies of the same label are a requirement when multiple syringes from a single multi-dose vial are prepared in advance, or if multiple syringes of the same medication are required for a single case. Safe Label System's optional Copy Mode feature enables a user to print up to nine (9) copies of the original label of Individual medications as approved by pharmacy from a single scan of a vial or ampoule.




Fentany  
10 mcg/mL



Copy Mode

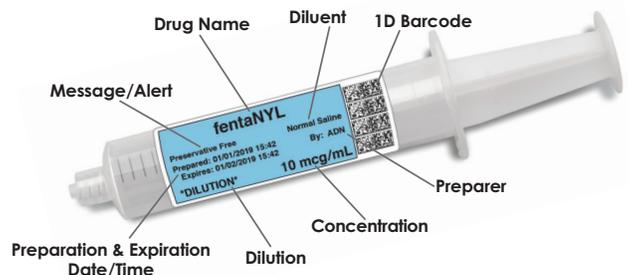

Safe Label System

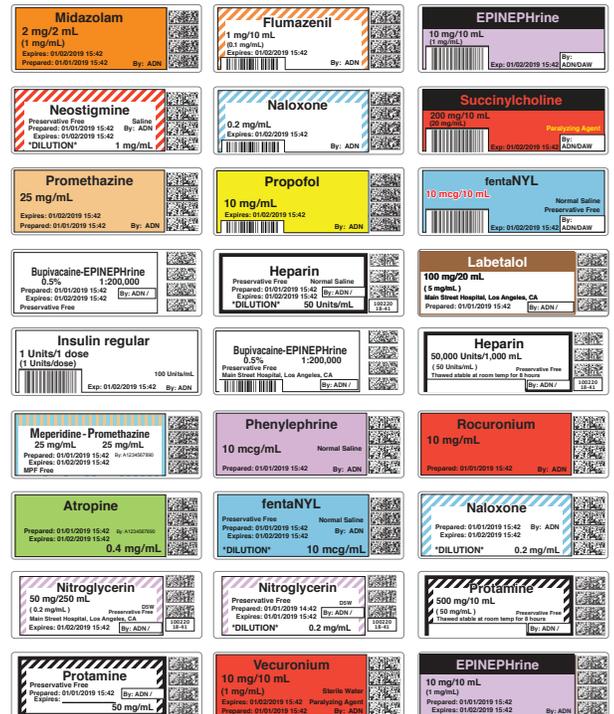

**In either mode**, Safe Label System electronically verifies each and every vial and/or ampoule in hand as a first line of defense to prevent medication administration errors, such as with dangerous look-alike drugs and hard to read vials and ampoules. Every easy-to-read, full-color label contains all of the elements required by The Joint Commission NPSG.03.04.01\* and meet the intent of ISO 26825, ASTM 4774 standards and ASA guidelines. TJC-compliant labels are presented, ready to apply to the medication immediately.



We bring the future into focus  
17991 Englewood Drive  
Middleburg Heights, OH 44130 USA  
+1.440.243.1198  
+1.440.243.1334 Fax  
info@codonics.com  
www.codonics.com

Call Codonics today at **800.444.1198** or visit [www.codonics.com](http://www.codonics.com) for more information.

\*Prints diluent and dilution if required. All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents: [www.codonics.com/ip/patents](http://www.codonics.com/ip/patents)  
Printed in the U.S.A. Copyright © 2019 Codonics Inc. 6/2019



Sponsoring Donor Corporate Member