

# Codonics Horizon Operating Software

**Technical Brief**

---

## Version 2.2.1 Release Notes

### Summary

Horizon<sup>®</sup> 2.2.1 software provides support for proper edge blanking (border) position around the image on 8x10 DirectVista<sup>®</sup> media in disposable cassettes.

### Products Affected

Imagers with a serial number prefix 50C, 51C, or 52C are affected. Please contact your Codonics representative for further information.

The Imager Software Version can be determined using the Control Panel by viewing the Imager Info screen from the Utilities menu.

**NOTE:** Horizon Imagers with serial number prefix 50C, 51C, or 52C upgraded to Version 2.1.0 or higher software cannot be downgraded to any previous version.

### New Features (Adaptive Updates)

- ◆ None in this release.

### Product Improvements (Perfective Updates)

- ◆ None in this release.

### Defects Corrected (Corrective Updates)

- ◆ Corrected the edge blanking border position around the image. The border was printing off the side edge of 8x10 media.

### Known Common Issues

- ◆ Previous issues can be found in previous Release Notes.

### Known Uncommon Issues

- ◆ Previous issues can be found in previous Release Notes.

### Technical Support

If problems occur during software installation, please contact Codonics Technical Support at any time.

Phone: +1.440.243.1198

Email: [support@codonics.com](mailto:support@codonics.com)

Website: [www.codonics.com](http://www.codonics.com)

---

*Get it all with just one call*  
**800.444.1198**

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.  
Patents: [www.codonics.com/ip/patents](http://www.codonics.com/ip/patents)  
Copyright © 2020 Codonics, Inc. Printed in the U.S.A.  
Part No. 901-084-021.01.



17991 Englewood Drive  
Middleburg Heights, OH  
44130 USA  
+1.440.243.1198  
+1.440.243.1334 Fax  
Email [info@codonics.com](mailto:info@codonics.com)  
[www.codonics.com](http://www.codonics.com)

Codonics Trading Co, Ltd.  
317 XianXia Rd. Building B  
Unit 1412  
Changning Dist., Shanghai  
P.R. China, 200051  
86-21-62787701  
86-21-62787719 Fax

Codonics Limited KK  
AQUACITY 9F,  
4-16-23, Shibaura  
Minato-ku, Tokyo,  
108-0023 JAPAN  
81-3-5730-2297  
81-3-5730-2295 Fax